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1000 mL

refer to the label
consulter l'étiquette

This device shall only be used by physicians and clinical staffs.

Sterile provided with package unopened or undamaged.

Re-use of this product will cause the risk of cross infection and unpredictable health threat.

This product is not made with natural rubber latex.

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Directions for use

1. Open the sterile pouch aseptically.
2. Inspect the vacuum indicator is at "MAX" line. If not, do not use this product.
3. CLOSE CONNECTING TUBE CLAMP FIRST! Attach Male Luer Connector of the connecting tube (if provided) to Female Luer of patient drainage device.
4. After checking all connections are secure, open the white slide clamp on the bottle to activate vacuum.
5. Open connecting tube clamp to begin drainage procedure.
6. Repeat steps 1 to 5 with a new bottle if vacuum indicator is at "MIN" line, or bottle is almost full and further drainage is required.
7. Vacuum level is stable for 3 years if properly stored.

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PACIFIC HOSPITAL SUPPLY CO., LTD.

Head office : 4F, No. 160, Daye Rd, Beitou Dist., Taipei 112, Taiwan

Factory: No. 8, Tongke 2nd Rd., Tongluo Township, Miaoli 366, Taiwan

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Made In Taiwan / Fabriqué à Taiwan

Distributed Exclusively in North America

by **MedQuest Medical Inc.**

www.medquestmedical.com